

LAB PLASTER



Type II Model Plaster

Lab Plaster is a fine, white general-use plaster. It is recommended for models, flasking and articulating.

Available colors: white

Available formulas: regular, fast set

For dental use only.

Typical Material Properties*

Distilled Water/Powder Ratio	Compression Strength (Dry)	Initial Setting Time	Setting Expansion
47cc/100g	3,500 psi	Regular: 6-8 minutes Fast Set: 4-6 minutes	0.25%

*These results are based on the testing methods, frequency and procedures of Ransom & Randolph or its approved suppliers. The levels referenced herein are only for general guidance and do not constitute a firm specification.

Application Instructions

Manual Mixing

1. Add 100 g powder to 47 cc water in a clean, dry mixing bowl.
2. Soak for 30 seconds.
3. Vigorously spatulate for 30 seconds.

Mechanical Mixing

1. Add 100 g powder to 47 cc water in a clean, dry mixing bowl.
2. Soak until powder is wet.
3. Mix for 20 seconds, 200 to 300 turns of the blade.

Storage

Gypsum materials are subject to deterioration when exposed to the atmosphere. Store this product at 59-86 °F (15-30 °C). Keep container tightly closed once seal is broken to protect contents from moisture and high humidity.

Caution

Inhalation of powder may be hazardous to your health.

As the conditions or methods of use are beyond our control, we do not assume any responsibility and expressly disclaim any liability for any use of this product. Information contained herein is believed to be true and accurate but all statements or suggestions are made without warranty, expressed or implied, regarding accuracy of the information, the hazards connected with the use of the material or the results to be obtained from the use thereof. Compliance with all applicable federal, state, and local regulations remains the responsibility of the user. All potential liability related to the sale and use of this product is limited to the cost of the particular goods sold in their respective transactions.



RANSOM & RANDOLPH

3535 Briarfield Boulevard, PO Box 1570 | Maumee, OH 43537 USA
800.253.4502 | 419.865.9497 | 419.865.9997 (FAX) | www.ransom-randolph.com



Issue Date: October 5, 2020

Investing with Innovation™